



Do not remove this information from dispenser box.

The Gallant Disposable Prep Razor comes in a convenient dispenser box containing 50 x cover protected prep razors.

- Finger-tip control through grip ridges on handle.
- Handle positioned in front of blade, requiring only a gentle pull.
- Optimal view of shaving area.
- PTFE and CrN coated razor blade.
- Razor blade facet specifically developed for a smooth cut.

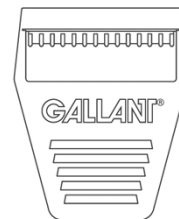



 CAUTION – Contains sharp razor blades


 Instructions for use:


1. Gently pull off the protective cover.
2. Place the razor over the designated shaving area. Do not apply any pressure. Gently move the razor over the skin.
3. For disposal, hold the razor upright against a flat surface and gently press until handle snaps. The razor is safely disabled.
4. The handle can be recycled and razor blade head can be disposed in a medical safety container.

 EU Declaration of conformity.

 Medical device.


 UDI: 7391100000018

 Commercial Product code: 120-354 / Dispenser Box with 50 units.

 Serial number dispenser box.

 Batch/ LOT code, see dispenser box.

 Do not use if broken.

 Do not reuse.

 Non-sterile.

 Date of Manufacture. See dispenser box.

 Expiry date. See dispenser box.

 Store your Gallant disposable razor in a dry place.

 Keep out of sunlight.

 Temperature limitation. Store between 5°C / 41°F - 30°C / 86°F.

 Manufacturer: Konsultbyrå Process Construction AB, Tågarpvägen 2 27293 Tommarp, Sweden  
www.gallant-sweden.com • vkl@gallant-sweden.com • ISO 13485 certified company

Gallant Disposable Prep Razor medical device registrations:

- EU MDR class 1, EU SRN: SE-MF-000005617
- US Food and Drug Administration, reg.no.: 3009143236
- MHRA UK Reg.No.: 2021121301226985
- Australia ARTG ID reg.no: 326611

Do not re-use on other patients. Risk for cross contamination and infection.  
Any serious incident occurred in relation to the product should be reported to the manufacturer.